


# SPECIMEN COLLECTION INSTRUCTIONS

## SurePath™ Pap Specimen


### Option 1

BD SurePath™ Test sample collection with Rovers® Cervex® brush.\*

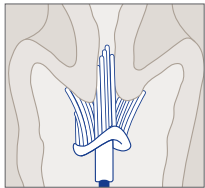

#### 1. Collect:

Collect the cytology sample using a broom-like device with detachable head. Follow manufacturer's collection instructions for detachable-head device(s).

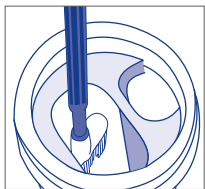

#### 2. Drop:

Use the interior rim of the new BD SurePath™ Collection Vial to pull off the head of the broom-like device into the large opening in the collection vial.



#### 3. Send:


Place the cap on the vial and tighten. Send the BD SurePath™ Collection Vial to the lab for processing.

### Option 2

BD SurePath™ Test sample collection with spatula and/or endocervical brush with detachable heads.\*\*

#### 1A. Collect:

Insert the contoured end of the Pap Perfect® plastic spatula and rotate 360° around the entire exocervix.

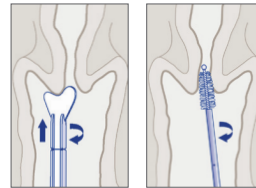

#### 2A. Drop:

Snap the device handle at the red scoring line and drop the detachable head of the device into the BD SurePath vial.



#### 3A. Next:

Place cap on vial; do not tighten cap until "Step 3B Send." Go to Cytobrush® Plus GT "Step 1B. Collect."



#### 1B. Collect:

Insert Cytobrush® Plus GT into the endocervix until only the bottom-most bristles are exposed at the os. Slowly rotate 1/4 to 1/2 turn in one direction. To reduce unnecessary bleeding, do not over-rotate brush.

#### 2B. Drop:

Snap the device handle at the red scoring line and drop the detachable head of both devices into the larger opening of the BD SurePath vial.

#### 3B. Send:

Place the cap on the vial and tighten. Send the BD SurePath vial to the lab for processing.

1. Bigras G, et al. J Low Genit Tract Dis. 7(3), 2003:168-174

\*See Rovers Cervex-Brush product insert for complete directions for use.

\*\*See SurePath Sample Collection Kit product insert for complete Directions for Use.

